



INTEFAB

DROP FOOT SYSTEM

XFT - 2001E

P O S S I B L E C A N D I D A T E S

Individuals with Central Nervous System Disorders, for Example:

- Incomplete Spinal Cord Injury (iSCI)
- Traumatic Brain Injury (TBI)
- Stroke (CVA)
- Multiple Sclerosis (MS)
- Cerebral Palsy (CP)
- Familial Hereditary Spastic Paraplegia (FSP)

Non-Candidates for the WalkAide® II

Individuals with Peripheral Nerve Damage, for Example:

- Secondary complications of back, hip or knee surgery
- Traumatic accidents to the leg
- Sciatica
- Peripheral neuropathy
- Spinal stenosis
- Post-polio syndrome
- Guillain-Barre





INTEFAB

DROP FOOT SYSTEM

XFT - 2001E

Indications, Contraindications, PRECAUTIONS & considerations

- **Indications:** Upper motor neuron (UMN) lesions
- **Contraindications, Warnings and Precautions:**
- Lower motor neuron (LMN) lesions
(ie. peripheral nerve damage from injury, disease or surgery, GB, CMT and Polio)
- * Pacemakers
- * Seizures
- * Pregnancy
- Malignant tumors in the leg
- Deep vein thrombosis
- Inappropriate response to stimulation
- Chronic skin conditions or poor skin integrity





INTEFAB

DROP FOOT SYSTEM

XFT - 2001E

Indications, Contraindications, PRECAUTIONS & considerations

- **Other Considerations:**

- Presence of other electrically controlled devices
- Baclofen pumps, deep brain stimulators or indwelling control devices
- Morbid obesity (fat tissue is too insulating for stim to be effective)
- Unstable cardiovascular status/HTN
- Severe balance deficits/vestibular issues
- Severe sensory deficit
- Chronic infectious disease
- Neurogenic pain syndromes

